


## EFFICIENT, EFFECTIVE, ESSENTIAL

ezGLIDE® is a latex free transfer sheet designed to reduce friction when repositioning or laterally transferring patients. ezGLIDE® is disposable and designed for single-patient use. It is an essential element in reducing the risk of hospital-borne infection.

- **Decreases job-related injuries – Eliminates lifting**
- Increased infection control
- Efficient operating room turnover
- May move from room to room with patient
- 4 mil. polyethylene designed to minimize the coefficient of friction
- Effectively eliminates need for transfer boards
- Single-patient use product reduces cross contamination
- Recyclable/Disposable
- Lightweight convenient design easily bridges two surfaces
- ezGLIDE® transfer sheet may be used with static sensitive electronic devices.

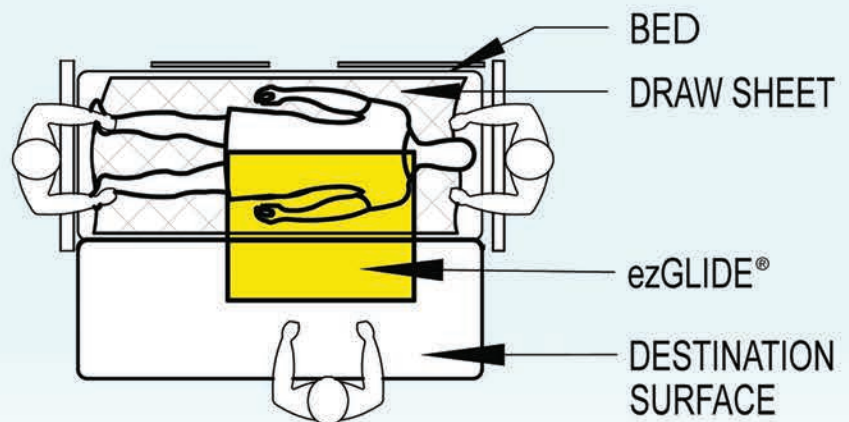

13316 A STREET • OMAHA, NEBRASKA 68144

P: 800.782.3742 • F: 888.323.8565 • INFO@SLOANCO.COM • WWW.SLOANCO.COM

# ezGLIDE® Innovative Patient Transfer System



Using the log rolling technique, slide the ezGLIDE® patient transfer sheet under the patient's draw sheet (if applicable). The ezGLIDE® should be inserted to the patient's midline, between their shoulder and hip.



Confirm that the origin and destination surfaces are stable and all wheels are locked.



Transfer the patient by pulling on the draw sheet (if applicable) and/or pushing on the patient's shoulder and hip.



Using the log rolling technique, remove the ezGLIDE® patient transfer sheet.



13316 A STREET • OMAHA, NEBRASKA 68144

P: 800.782.3742 • F: 888.323.8565 • INFO@SLOANCO.COM • WWW.SLOANCO.COM